visio.sil

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Instructions for use

visio.sil

Before using the product, please read these instructions for use carefully.

1. Indication range

visio.sil is an addition-curing silicone which was especially developed for the use in the pre-impression technique with UV-curing materials. Thanks to the transparency, the material is particularly suited for processing techniques used to achieve high-quality aesthetic restorations. Above all in conjunction with the visio.lign veneering system, swift and economic working is ensured by curing through the matrix material.

1.1 Contraindication

visio.sil is not suitable for oral use.

2. Processing

visio.sil can be immediately applied out of the Automix cartridge system. The material hardens within 4 minutes.

3. Storage

visio.sil should be stored between 8° C and 21° C. If the material is stored below a room temperature of 21° C, the setting reaction is delayed if a longer processing time span is desired.

4. Safety information

visio.sil may only be used by dental technicians and trained dental staff who are experienced in the handling of silicone materials. visio.sil is exclusively intended for the use in dental-technical procedures for completion work.

Avoid contact with skin and eyes and do not swallow the material.

5. Additional information

The information contained in these instructions for use is always updated according to the latest knowledge and experience. Therefore we recommend to read the instructions for use again before using a new package. Any recommendations concerning the use are based on bredent's experience and tests and can only be considered to provide approximate values. Our products are subject to continuous further development. Therefore we reserve the right to perform changes in the design and composition.

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